CoolFlow SM-03-DTX

Information Booklet

(Re-filling Instructions
& Safety Information)

www.spectrummedical.com

SUM-4000002_v2a

© 2020

Copyright in this document belongs to Spectrum Medical Ltd. The information contained herein which is confidential belongs to Spectrum Medical Ltd. This document is not to be used for any purpose whatsoever without the prior written consent of Spectrum Medical Ltd.

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

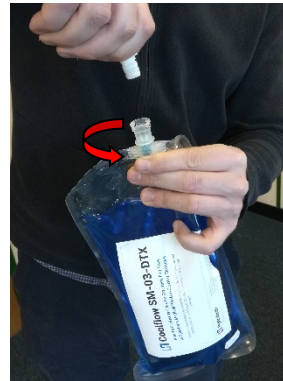
Refilling instructions

The Cold Storage Charger is filled through the filling connector which is located at the back of the Termotek charging unit on the right-hand side, behind a magnetic flap.

1. Open the filling flap.
2. Remove cap from the replenishment bag.

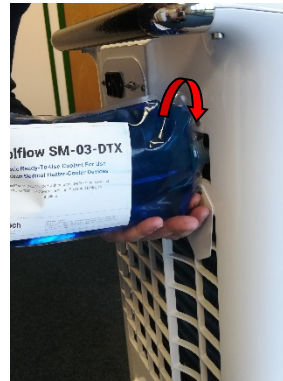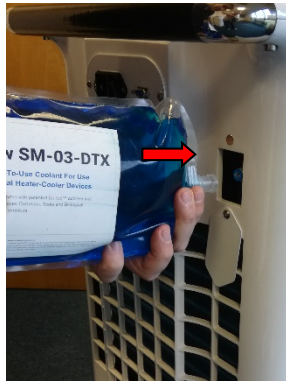
3. Press the bag connector onto the filling connector and turn clockwise to lock into position.

4. Begin re-filling by holding the bag above the connector point.

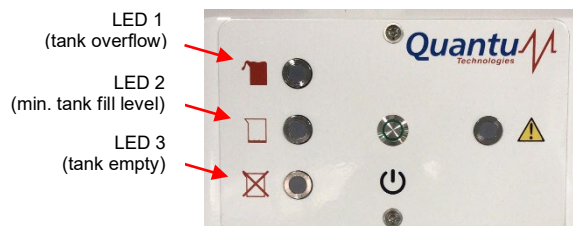5. Continue re-filling until the LEDs (2 and 3) go out. LEDs are located on the front display.
6. To check the fill level, use the fill level indicator located on the left side of the unit.

Note: If the pilot LED (1) lights up, the tank is overfilled. Drain the medium from the tank via the drainage point located at the bottom of the unit.

Refer to the user guide supplied with the Cold Storage Charger for further information on drainage.
Refer to section 7 of this booklet for fluid disposal information.

1 Section 1: Identification of the substance/mixture and of the company/undertaking

1.1 Product identifier

Product Name Coolflow SM-03-DTX

1.2 Relevant identified uses of the substance or mixture and uses advised against

Identified uses Uses Non-toxic ready-to-use coolant with inhibitor functions designed for use in Spectrum Medical heater-cooler devices.
advised against This product is not recommended for any industrial, professional or consumer use other than the identified uses above.

1.3 Details of the supplier of the Safety Information Sheet

Supplier Spectrum Medical Limited
Harrier 4, Meteor Business Park,
Cheltenham Road East,
Gloucester, GL2 9QL
United Kingdom
+44 (0) 1242 650120

2 Section 2: Hazards identification

2.1 Classification of the substance or mixture

Classification - Regulation (EC) No. 1272/2008 (CLP)

Physical and chemical hazards Not classified as a physical or chemical hazard
Human health Not classified as a health hazard
Environment Not classified as an environmental hazard

In accordance with Article 9 and Article 12 of Regulation (EC) No. 1272/2008 (CLP), Coolflow DTX has been assessed as a mixture where adequate and reliable scientific information demonstrates the occurrence of synergistic and antagonistic effects that renders the mixture non-toxic as determined by an EPA certified laboratory with an LD50, oral, rat >15,000 mg/kg bw.

Glutaraldehyde solution (50%): 0.034%.

3 Section 3: First Aid Measures

3.1 Description of first aid measures

General Information When safe to do so remove the victim from the source of exposure giving consideration as to whether this may cause further discomfort to the victim.
Inhalation Move the affected person to fresh air at once. Keep warm in a position comfortable for breathing. If breathing becomes difficult, properly trained personnel may assist the victim by supplying oxygen to ease breathing. Get medical attention if any discomfort continues.
Ingestion Move the affected person to fresh air and keep warm in a position comfortable for breathing. Rinse mouth thoroughly with water. Get medical attention if any discomfort continues.
Skin Contact Remove contaminated clothing immediately and wash skin with soap and water. Get medical attention if any discomfort continues.
Eye Contact Rinse immediately with plenty of water. Remove any contact lenses and open eyelids wide apart. Continue to rinse for at least 15 minutes. Get medical attention if any discomfort continues.

3.2 Most important symptoms and effects, both acute and delayed

General Information The following symptoms are listed in case of exposure to the 100% neat product.
Inhalation Ingestion Inhalation of vapours may cause mild irritation of the upper respiratory tract. May cause discomfort to the stomach if swallowed.
Skin Contact
Eye Contact Prolonged and repeated contact may cause mild irritation of the skin.
Direct eye contact may cause reddening of the eyes.

3.3 Indication of any immediate medical attention and special treatment needed

No specific recommendations given, but first aid may still be required in case of accidental exposure, inhalation or ingestion of this chemical. If in doubt, get medical attention promptly and present a copy of this Safety Information Sheet.

4 Section 4: Handling and storage

4.1 Precautions for safe handling

Avoid spilling and contact with the skin and the eyes as well as direct inhalation of sprays and mists.
Provide good ventilation.
Do not eat, drink or smoke in work areas and wash hands after handling this product.

4.2 Conditions for safe storage, including any incompatibilities

Store in tightly-closed, original containers.
Keep separate from food, feedstuffs, fertilisers and other sensitive material.
Do not store near heat sources or expose to high temperatures.
Keep away from heat, sparks and open flame.

4.3 Specific end use(s)

Absorb spillage with inert, damp, non-combustible material, then flush the contaminated area with water.
Collect and place in suitable waste disposal containers and seal securely. For waste disposal, see Section 7.

5 Section 5: Exposure controls/personal protection

Technical procedures

| | |
|---|---|
| Engineering measures | Methods to prevent or control exposure are preferred. Provide adequate ventilation to minimise the risk of inhalation of sprays and mists. |
| Hygiene measures | Handle in accordance with good industrial hygiene and safety practices. Wash hands after handling this product and at the end of each work shift. Routinely wash work clothing and personal protective equipment to remove possible contaminants. |
| Respiratory equipment | If ventilation is inadequate, suitable respiratory protection must be worn. |
| Hand protection | PVC/butyl rubber/neoprene gloves are recommended. |
| Eye protection | Wear approved chemical goggles or face shield. |
| Skin Protection | Wear rubber apron or protective clothing in case of contact. |
| Other Protection | Wear suitable protective clothing/footwear as protection against splashing or contamination. |
| Thermal Hazards Environmental Exposure Controls | No specific measures required. Product not classified as an environmental hazard - no specific environmental exposure controls required. |

6 Section 6: Physical and chemical properties

| | | | |
|---|--|---|------------------|
| Appearance / Colour: | Clear, slightly viscous liquid / Blue | Vapour pressure: | 13 Pa @ 25°C |
| Odour: | Non-pungent but characteristic aroma | Vapour density (air = 1): | 2.2 |
| pH: | 7.0 - 8.0 | Relative density of the mixture: | 1.01 - 1.06 |
| Melting point / Pour point: | -22°C | Solubility: | Soluble in water |
| Initial boiling point: | >103°C | Auto-ignition: | >400°C |
| Flash point: | >115°C | | |
| Flammability: | Product is not classified as flammable | | |
| Flammability / explosion limits: | Upper limit: 15% Lower limit: 3% | | |

7 Section 7: Disposal considerations

General information

Waste to be treated as controlled waste. Disposal to licensed waste disposal site in accordance with Local Waste Disposal Authority.

Disposal methods

Dispose of waste and residues in accordance with local authority and/or local sewage treatment plant requirements.

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