

INSTRUCTIONS FOR USE

VenArt Cardiac Output Monitoring Platform



ENGLISH

Mespere
LIFE SCIENCES

Table of Contents

Section 1: Introduction	4
About this Manual	4
Paper Copy.....	4
Download.....	4
Previous Versions.....	4
Description	4
Indications for Use	4
Contraindications	4
Section 2: Safety Information	5
Warnings	5
Cautions	6
Section 3: Setup	7
Installation	7
Display Software Setup	7
Turning the Computer On and Off:	7
Monitor Display	8
VenArt Cardiac Output Sensors	10
VO 100 Jugular Venous Oximetry Sensor and Isolator	10
Sensor Adhesive	10
Pulse Oximeter	11
Section 4: Platform Operation	12
Setup for VenArt Cardiac Output Monitoring Platform	12
Obtaining Optimal Jugular Venous Waveform (i.e. adjusting patient incline angle)	13
Computer Functions	14
Platform Messages	17
Patient Environment	17
Platform Trend Display	18
Reading the Trend Data Display	18
Clearing Trend Information	18
Essential Performance	18
Sensor Performance Considerations	18

Factory Default Settings	18
Display Software	18
Section 5: Cleaning, Disinfection, Maintenance, and Disposal	19
Cleaning and Disinfection	19
Maintenance	19
Inspecting the Equipment and Accessories	19
Returning the System for Repairs and/or Service	19
Disposal	19
Section 6: Troubleshooting	19
For Technical Assistance	20
Section 7: Specifications	21
Performance	21
Electrical	21
Environmental	22
Physical Characteristics	23
Compliance	23
Warranty	23
Manufacturer’s Declaration	23
Section 8: Appendices	28
Appendix 1: List of Abbreviations	28
Appendix 2: List of Symbols	29
Appendix 3: List of VenArt Cardiac Output Components	31

Section 1: Introduction

About this Manual

Paper Copy

If a paper copy of this manual is needed, please contact your distributor or Mespere LifeSciences directly.

Download

The electronic version of this manual is available at <http://www.mespere.com/resources.html>. Please contact your distributor or Mespere LifeSciences directly to obtain the login information to download this manual.

Previous Versions

Please contact Mespere LifeSciences directly to obtain previous versions of this manual.

Description

The VenArt Cardiac Output Monitoring Platform is a software. The VenArt System is capable of continuously displaying the following parameters:

- Cardiac Output (CO);
- Stroke Volume (SV);
- Stroke Volume Variation (SVV);
- Jugular Venous Oximetry (SjvO₂); and
- Arterial Oximetry (SpO₂).

Indications for Use

Health Canada/CE Marking:

The VenArt Cardiac Output Monitoring Platform is indicated for displaying hemodynamic parameters of jugular venous oximetry and pulse oximetry. It also displays the calculated results of cardiac output, stroke volume, and stroke volume variation based upon Fick's Principle. The VenArt Cardiac Output Monitoring Platform may be used in all settings in which critical care is provided.

Contraindications

- Do NOT use the VenArt Cardiac Output on patients:
 - Undergoing photodynamic therapy (PDT).
 - Known to have allergic reactions to medical-grade adhesive tape.
 - With central vein stenosis.

Section 2: Safety Information

Warnings

WARNING



- Do NOT use the VenArt Cardiac Output unless on the order of a physician.
 - The VenArt Cardiac Output is intended for use by healthcare professionals only.
 - As per Standard IEC 60601-1 Section 11, it has been disclosed that the maximum temperature of the applied parts for the VenArt Cardiac Output is 43.2 degrees Celsius. It is recommended that the VenArt Cardiac Output is not left running on the patient when it is not in use so that the applied parts will not reach the maximum temperature. Caution must be taken for patients who are taking medication that would make their skin more susceptible to damage from sun or light.
 - The VenArt Cardiac Output may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as reorienting or relocating the VenArt Cardiac Output or shielding the location.
 - To avoid the risk of electric shock, the DM-10-S Power Supply (PS-03) must be connected to an AC outlet with protective earth.
 - Do NOT re-use sensor adhesives. Doing so may increase the risk of infection or cross-contamination.
 - Always unplug the system before cleaning. Refer to **Section 5: Cleaning, Disinfection, Maintenance, and Disposal** for more information.
 - Any changes or modification to this equipment not expressly approved by Mespere LifeSciences Inc. may cause bodily harm and void your authority to use this equipment.
 - The use of accessories and cables other than those specified, with the exception of accessories and cables qualified and sold by Mespere LifeSciences Inc. may result in increased emissions or decreased immunity of the equipment and may cause the VenArt Cardiac Output to be non-compliant with the requirements of IEC 60601-1-2:2014.
 - Pins of connectors should not be touched and that connections should not be made to these connectors unless ESD precautionary procedures are used.
 - The operator should NOT touch the computer and the patient at the same time.
 - The VenArt Cardiac Output should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, the equipment should be observed to verify normal operation in the configuration in which it will be used.
-

Cautions

CAUTION



- Read and understand this entire manual carefully before using the VenArt Cardiac Output.
 - The VenArt Cardiac Output needs precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in **Section 7: Specifications**.
 - Portable and mobile RF communications equipment may affect the VenArt Cardiac Output.
 - Do NOT sterilize any part of the VenArt Cardiac Output using any process.
No part of the VenArt Cardiac Output is to be sterilized. Cleaning must be performed as described in **Section 5: Cleaning, Disinfection, Maintenance, and Disposal**.
 - A cable connects the VenArt Cardiac Output sensor and isolator to the computer.
Do NOT twist or pull on this connection.
Do NOT allow this cable to become tangled in other equipment such as bed frames.
-

Section 3: Setup

Installation

Before the VenArt Cardiac Output can be used in a clinical setting, it should be inspected and properly setup.

Display Software Setup

The display software can be pre-loaded by distributor or Mespere LifeSciences to a computer. If the display software is not preloaded, please refer to the Display Software Installation document.

Procedure

1. Download VenArt Cardiac Output Installer file provided to you via Dropbox or by a Mespere LifeSciences representative.
2. Plug an external hard drive, thumb drive, or USB drive into your computer.
3. Place the VenArt Cardiac Output Installer file onto the external hard drive, thumb drive, or USB stick.
4. Plug external hard drive, thumb drive, or USB drive into the Computer. A pop-up window should appear showing the contents of this drive.
5. Open the VenArt Cardiac Output Installer file.
6. Allow the VenArt Cardiac Output Installer file to make changes to your computer by clicking "Yes".
7. Click "Install" on the VenArt Cardiac Output Installer Setup Wizard.
8. Once the VenArt Cardiac Output Installation is complete press "Finish". VenArt Cardiac Output Display Software is now installed on the computer.

Turning the Computer On and Off:

Once the computer is turned on the Mespere home screen will appear. If the display software is not pre-loaded please refer to the Display Software Installation document.



Procedure

1. Turn the computer on by pushing the power button.
2. Once the Mespere home screen appears, the Platform is ready for use.
3. Turn the monitor off by pushing the virtual power button on the home screen

Monitor Display

The VenArt Cardiac Output displays the plethysmographic waveform, S_{jv}O₂, SpO₂, BVI, CO, SV, and SVV values in real time.



Control/Indicator	Description
Venous Oxygen Saturation (S _{jv} O ₂) Measurement	Shows the numerical value of jugular venous oxygen saturation (S _{jv} O ₂) in percentage (%).
Pulse Strength Index (PI)	Indicates the strength of the pulse signal. When the signal is less than 1, indicating a weak signal, the number will be displayed in red and the waveform will not be visualized. If the signal is greater than 1, indicating a strong signal, the number will be displayed in white and a defined waveform will be visualized. Please note the PI will not affect the S _{jv} O ₂ only the visualization of the waveform.
Blood Volume Index (BVI)	Blood to tissue volume ratio of the regional tissue beneath the sensor. If a reading is 10% then that means that 10% of the tissue volume is filled with blood.
Heart Rate	Number of heart beats per minute
Arterial Oxygen Saturation (SpO ₂) measurement	Shows the numerical value of arterial oxygen saturation (SpO ₂) in percentage (%)
Stroke Volume Variation (SVV)	The change in stroke volume with each heartbeat in percentage (%). The data will be displayed after 30 seconds of readings.
Stroke Volume (SV)	The amount of blood ejected from the left ventricle with each heartbeat in millilitres (mL)
Cardiac Output (CO)	The amount of blood the heart pumps through the circulatory system in a minute (L/min)
Cardiac Index (CI)	The amount of blood the heart pumps through the circulatory system in a minute in relation to the patient's body surface area (L/min/m ²)
Patient Parameters	Input patient's weight (W) in kg, height (H) in cm, and hemoglobin concentration (Hg) in g/dL.
Trend Plot	The data trends for the selected parameter.
Arterial Waveform	The arterial waveform detected by the pulse oximeter.
Right/Left Venous Waveform	The plethysmographic waveform detected by the right or left VO 100 sensor.


Control/Indicator	Description
ID – Patient Folder	Once a patient ID is entered, a patient folder will be created using this ID and all patient data will be saved in this folder. If a patient ID is not entered the files will be saved in the temp folder using the current time stamp as the file name.
New Measurement	Pressing this button will clear the stopped data on the screen
Start	Pressing this button will start a new monitoring session
Stop – Auto Save	Press this button to stop the current monitoring session. Once the monitoring session is stopped an excel file, screenshot, marked events, and summary pdf will automatically be saved to the patient's folder.
Event Marker	Mark an event by displaying a vertical line on the trend window. Enter event details and/or press OK once complete. Marked events can be reviewed at any time under the Review – Trend window.
Trend Toggle	Change the displayed data trend (SO ₂ , CO/CI, SV, or BVI)
Fluid Challenge	Three different reversible fluid challenge protocols; Passive Leg Raise (PLR), Bolus, and Tilt Table, in order to determine fluid responsiveness.
Review	Trend: view the complete S _{jv} O ₂ , SpO ₂ , BVI, CO, CI, SV, and SVV trend summary and marked events from the entire monitoring session. Press Save Review on the top toolbar to save a pdf summary and screenshot to the patient's folder Compare Snap Shot: Select two Snap Shot files to compare the data and create a comparison report. Press Save Comparison on the top toolbar to save a pdf summary to the patient's folder.
Snap Shot	Create a report based on the average of the last 30 seconds of data. An excel file and pdf summary will automatically be saved to the patient's folder.
Export Data	Export previously saved patient files such as Snap Shots, Review Summaries, Excel Files, and Screenshots to a removable drive. Patient data is saved by patient ID. Locate the patient's folder and copy desired files to removable drive. If a patient ID was not entered, the patient's data files will be located in the temp folder and can be identified using the time stamps as the file name.
Help	Help Center
Settings	Sensitivity: if the EJV or IJV is not visible, select high sensitivity. If visible keep at default. Calibration: optional based upon users preference Data Recording Frequency: select whether data is stored every 1 or 30 seconds in the data files
Minimize	Minimize the CO/SV application to return to the homescreen
Exit	Exit the CO/SV application to return to the homescreen

VenArt Cardiac Output Sensors

The VenArt Cardiac Output Monitoring Platform uses 1 or multiple Mespere LifeSciences' VO 100 Jugular Venous Oximetry sensors and a pulse oximeter.


VO 100 Jugular Venous Oximetry Sensor and Isolator

The VO 100 Jugular Venous Oximetry Sensor consists of a sensor cable and sensor head. The sensor head contains the LEDs and PDs. The isolator consists of a cable connector, an isolator, flexible cable, and USB connector. The isolator connects the VO100 Jugular Venous Oximetry sensor to the computer or USB hub.

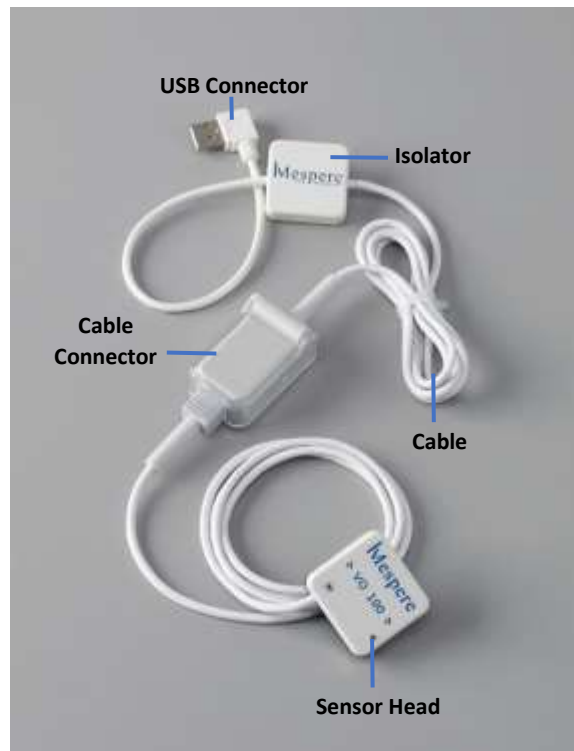

Procedure

1. Open the clear case on the isolator cable connector.
2. Plug the cable connector from the VO 100 Jugular Venous Oximetry sensor into the cable connector of the isolator.
3. Close the clear case on the cable connector.

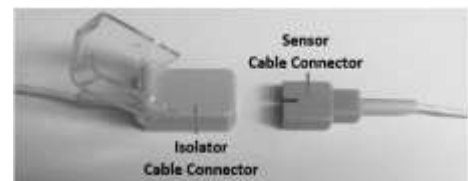

Sensor Adhesive

The VO 100 Jugular Venous Oximetry sensor adhesives are single-use, and use a medical-grade adhesive and foam.

The sensor adhesive clips onto the VO 100 Jugular Venous Oximetry sensor head and is used to adhere the VO 100 Jugular Venous Oximetry sensor to the patient.



WARNING



- The VO 100 Jugular Venous Oximetry sensor adhesives are intended for single-use only.
 - Do NOT re-use the VO 100 Jugular Venous Oximetry sensor adhesives. Doing so may increase the risk of infection or cross-contamination.
-

Pulse Oximeter

The following pulse oximeters are compatible with the VenArt Cardiac Output Monitoring Platform:

- Nonin 8000S
- Contec CMS50E

For additional information please refer to the Instructions For Use provided with the Nonin or Contec sensors.

Nonin 8000S Procedure

The Nonin 8000S pulse oximeter consists of a XPod and pulse oximeter. The XPod connects the pulse oximeter to the computer or USB hub.

1. Plug the pulse oximeter sensor into the XPod.
2. Plug the XPod into the computer or USB hub.

Contec CMS50E

The Contec CMS50E pulse oximeter consists of a data line and pulse oximeter.

1. Plug the data line into the pulse oximeter.
2. Plug the data line into the computer or USB hub.

Section 4: Platform Operation

Setup for VenArt Cardiac Output Monitoring Platform

Monitor Setup

1. Choose the CO/SV application from the homescreen.
2. Plug the VO 100 sensor into the monitor.
3. Plug the Pulse Oximeter sensor into the monitor.

Sensor Setup

4. Snap a disposable adhesive onto the sensor head with the yellow dot adjacent to the logo.

Sensor Placement

5. Identify the patient's right internal or external jugular vein (IJV or EJV).
6. Position the right sensor (R) so that the arrow points towards the patients head and the midline of the sensor is directly over the IJV or EJV.
7. The right sensor is suggested for CO/SV calculations
8. Repeat with the left sensor (L) if conducting bilateral readings.
9. Place the pulse oximeter on the patient's index finger. If using the Contec CMS50E pulse oximeter, turn it on by pushing the white button on top of the sensor.

Patient Setup

10. Place the patient at a comfortable inclination (between 0 and 30 degrees is suggested).
11. Slightly tilt the patient's head to the left, and have the patient remain quiet and keep their neck relaxed.

Patient Parameters

12. Input the patient's height (H) in cm, weight (W) in kg, and hemoglobin concentration (Hg) in g/dL.

Settings

13. Press the Settings button on the monitor.
 - **Sensitivity:** if the EJV or IJV is not visible, select high sensitivity. If visible keep at default.
 - **Calibration:** optional based upon users preference.
 - **Data Recording Frequency:** select whether data is stored every 1 or 30 seconds in the data files.
14. Press OK.

Fluid Challenge

15. Press the Fluid Challenge button and choose the appropriate fluid challenge method; Passive Leg Raise (PLR), Bolus, or Tilt Table. Follow the instructions on the screen to complete the selected fluid challenge.

Passive Leg Raise (PLR)

16. Select the PLR method from the drop down menu.
17. Place the patient in a semi-recumbent position and press OK.
18. A 30 second countdown will appear on the screen.

19. Place the patient in a passive leg raise position and press OK.
20. A 30 second countdown will appear on the screen.
21. Once complete proceed to the Report.

Bolus

15. Select the Bolus method from the drop down menu.
16. Start administering a bolus challenge.
17. Once the bolus is complete press Stop.
18. Once complete proceed to the Report.

Tilt Table Method

15. Select the Tilt Table method from the drop down menu.
16. Place the patient in position 1 and once a baseline is achieved press OK.
17. Place the patient in position 2 and once a baseline is achieved press OK.
18. Once complete proceed to the report

Fluid Challenge Report

19. Once the selected fluid challenge is complete, a Report will be generated.
20. Press Save Report on the top toolbar to save a pdf report summary to the patient's folder.

Obtaining Optimal Jugular Venous Waveform (i.e. adjusting patient incline angle)

Recline or incline the patient until a clear waveform appear on the screen. The VO 100 sensor uses Near Infrared Spectroscopy (NIRS) technology to measure jugular venous waveform. The VO 100 sensor shines near infrared photons into the tissue containing jugular venous blood vessels via a sensor array. The sensor array analyzes the diffusely reflected photons and is then able to determine jugular venous waveform. In order to get a defined waveform, the sensor must detect the height of the venous blood column beneath the sensor. Changing the degree of inclination for the patient will help position the jugular venous blood column beneath the VO 100 sensor in order to achieve a stable waveform.

If PI is less than 1 and in red:

1. Ensure that the sensor is placed directly over the right or left external jugular vein.
2. Use the physician's judgement on the waveform

Computer Functions

ID – Enter Patient ID

1. Enter a patient ID into the white input box anytime throughout the monitoring session.
2. A patient folder will be created using this ID and all patient data will be saved in this folder. If a patient ID is not entered the file will be saved in the temp folder using the current time stamp as the file name.

New Measurement (New Meas.)

1. Press New Meas. to clear the previously stopped data showing on the screen.

Start

1. Press Start to begin a new monitoring session.

Stop – Auto Save

1. Press Stop to stop the current monitoring session.
2. Once the monitoring session is stopped an excel file, screenshot, and summary pdf will automatically be saved to the patient's folder.

Trend

1. Press the Trend button.
2. A dropdown menu will appear to change the displayed trend in the trend plot; CO, SjvO2/SpO2, SV, or BVI.

Event Marker

1. Press the Event Marker button.
2. Enter event details and press OK.
3. A vertical line will appear in the trend window marking an event.

Fluid Challenge – Passive Leg Raise, Bolus, or Tilt Table

1. Press the Fluid Challenge button.
2. A dropdown menu will appear.
3. Select the desired fluid challenge to proceed with.

Change Event Description

1. Press the Review button and select Trend.
2. The Review – Trend screen will appear. Press View Events from the top toolbar.
3. A pop-up window will appear with the marked events.

4. Click the event box and enter the appropriate description.

Review - Trend

1. Press the Review button on the monitor. A dropdown menu will appear and select Trend.
2. A window will appear with a complete SjvO₂, SpO₂, BVI, CO, and SV trend summary and marked events from the entire monitoring session.
3. Press Save Review on the top toolbar to save a pdf summary and screenshot to the patient's folder.

Review – Compare Snap Shot

1. Press the Review button on the monitor. A dropdown menu will appear and select Compare Snap Shot.
2. A pop-up window will appear. Select two Snap Shot files to compare the data and create a comparison report.
3. Press Save Comparison on the top toolbar to save a pdf summary to the patient's folder.

Stop – Auto Save

3. Press the Stop button located on the top toolbar once the monitoring session is complete.
4. A pop-up window will appear showing that an excel file, screenshot, marked events, and summary pdf will automatically be saved to the patient's folder.

Snap Shot

1. Press the Snap Shot button located on the top toolbar.
2. A report will be created based on the average of the last 30 seconds of data. An excel file and pdf summary will automatically be saved to the patient's folder.

Export Data – Snap Shots, Review Summaries, Excel Files, and Screenshots

1. Press the Export Data button on the top toolbar.
2. Locate the patient's folder. If a patient ID was entered, a folder was created using this ID and all patient data will be saved in this folder. If a patient ID is not entered the file will be saved in the temp folder using the current time stamp as the file name.
3. Plug in an external memory source to the USB port on the right side of the computer.
4. Click the file until a menu appears. Select Copy
5. Open the external memory source and hold the screen until a menu appears. Select Paste. The copied files will be transferred to the connected removable drive.

Deleting saved files

1. Press Export Data button on the top toolbar.
2. Locate the patient folder.
3. Click the file until a menu appears.
4. Select delete from the menu.

Checking Sensor Life

1. Plug the sensor into the USB port on the right side of the computer.
2. Press the Help button.
3. Press the sensor life button.
4. A pop-up window will appear indicating the number of hours remaining for that specific sensor.

Platform Messages

The guidance messages on the Display are shown in Table 1:

Table 1: Informative platform messages

Message	Cause and Resolution
“VO100 Sensor is disconnected”	<p>The VenArt Cardiac Output is detecting that the sensor is not connected to the computer.</p> <ul style="list-style-type: none"> - Ensure that the VO 100 sensor is correctly connected to the isolator. - Ensure that the isolator is correctly connected to the computer.
“VO100 Sensor Expired. Please Replace!”	<p>The VenArt Cardiac Output is detecting that the attached VO100 sensor has expired.</p> <ul style="list-style-type: none"> - Replace the connected VO100 sensor with a new one or one that hasn't expired
“Check VO100 Sensor Patch Adhesion”	<p>This may if the sensor has loose adhesion to the patient's neck.</p> <ul style="list-style-type: none"> - Ensure that the sensor is adhered to the patient's neck.
“Weak Pulse”	<p>The VenArt Cardiac Output is detecting that the PI is less than 1.0 therefore a defined waveform may not appear.</p> <ul style="list-style-type: none"> - Adjust the patient angle in order to position the taper of the jugular vein beneath the LEDs and PDs for a defined waveform.
“USB Pulse Oximeter Sensor is disconnected”	<p>The VenArt Cardiac Output is detecting that the pulse oximeter is not connected to the computer.</p> <ul style="list-style-type: none"> - Ensure that the pulse oximeter is correctly connected to the computer.
“Finger Out”	<p>The VenArt Cardiac Output is detecting that the patient's finger has been removed from the Pulse Oximeter.</p> <ul style="list-style-type: none"> - Ensure that the patient's finger is in the pulse oximeter
“Demo Mode”	<p>The VenArt Cardiac Output is running in demo mode.</p> <ul style="list-style-type: none"> - Press “Exit Demo Mode” or connect a sensor to exit the demo application.
“Please wait”	<p>The VenArt Cardiac Output is loading. Please wait until this message has closed, before continuing.</p>
“__ hours of usage has elapsed”	<p>The attached VO 100 sensor has been used for this many hours.</p>

Patient Environment

The VO100 sensor adhesives and the Pulse Oximeter are the applied parts of the VenArt Cardiac Output that will come in contact with the patient during normal use.

The sensors and sensor adhesive are components of the VenArt Cardiac Output that will be in the patient environment.

NOTE



The patient environment is defined as the area that is 1.5m from the patient in their bed or chair.

CAUTION



The isolator chip and the computer should be kept away from the patient environment.

Platform Trend Display

The history of the S_{ijv}O₂, SpO₂, BVI, CO, and SV trend are displayed on the computer.

Reading the Trend Data Display

The S_{ijv}O₂, SpO₂, BVI, CO, and SV trend are displayed on the computer, with the x axis representing number of data points and the y axis displaying the history S_{ijv}O₂, SpO₂, BVI, CO, and SV data. The current version of the VenArt Cardiac Output does not allow the user to display the S_{ijv}O₂, SpO₂, BVI, CO, and SV trend data in another unit.

Clearing Trend Information

All history VenArt Cardiac Output data is erased when a new measurement is started by pressing the Start button.

Essential Performance

The VenArt Cardiac Output has no essential performance characteristics.

Sensor Performance Considerations

CAUTION



Patients must not move excessively during measurements. Excessive talking and motions such as yawning, deep-breathing and the like should be discouraged while measurements are being taken in order to get defined waveforms.

Factory Default Settings

Display Software

The computer will include the VenArt Display Software (OS-VenArt) with software version VA.EN.01.00.046

Section 5: Cleaning, Disinfection, Maintenance, and Disposal

Cleaning and Disinfection

The VenArt Cardiac Output requires no preventive maintenance. Only regular cleaning and disinfection based on the institutional or jurisdictional guidelines is required. In the absence of guidelines, it is recommended that cleaning is performed after each use of the VenArt Cardiac Output.

WARNING



- Always unplug the sensors before cleaning.
- Allow the VenArt Cardiac Output to dry completely before re-connecting power.

CAUTION



- Do NOT immerse any part of the VenArt Cardiac Output in any liquid for any reason.
- Cleaning should be performed using institutional or jurisdictional guidelines.
- Do NOT subject any component of the platform to sterilization (e.g. ETO, steam). Doing so may damage the components of the VenArt Cardiac Output.

Maintenance

The VenArt Cardiac Output requires no preventive maintenance.

Inspecting the Equipment and Accessories

Prior to each use of the VenArt Cardiac Output, it is suggested that the user checks that all connections to the computer, isolator, and sensors are correct and secure. For instructions on how to make these connections, please refer to **Section 3: Setup**.

Returning the System for Repairs and/or Service

Contact your distributor for information on repairs and/or services for the VenArt Cardiac Output.

Disposal

Comply with local laws for disposal of VenArt Cardiac Output.

CAUTION



Product is to be taken to separate collection at end of product life. Do NOT dispose of the product as unsorted municipal waste.

Section 6: Troubleshooting

The following table provides basic guidance in the event of a problem with the VenArt Cardiac Output.

Table 2: List of Problems for Troubleshooting

Problem	Possible Causes	Possible Remedies
Computer will not power on	<ul style="list-style-type: none"> - No power 	<ul style="list-style-type: none"> - Check connection of the power supply to the computer. - Check connection of Power Supply to AC power (wall outlet). - Try alternate outlet.
Computer cannot communicate sensor	<ul style="list-style-type: none"> - Connection of sensor 	<ul style="list-style-type: none"> - Ensure sensors are connected correctly to the isolator and computer. - Try re-starting software by powering down the computer, then turn the computer on.
VenArt Cardiac Output does not display S _{ijv} O ₂ , SpO ₂ , CO, or SV value, or plethysmographic waveform	<ul style="list-style-type: none"> - Patient status - sensor placement 	<ul style="list-style-type: none"> - Be sure to follow the on-screen instructions for adjusting the patient angle. Refer to Section 3: Setup. - Check the sensor placement and orientation. - The VenArt Cardiac Output may not work with all patients due to anatomical or health status.
“Check VO100 Sensor Patch adhesion”	<ul style="list-style-type: none"> - Sensor has loose adhesion to the patient’s neck. 	<ul style="list-style-type: none"> - Ensure that the VO100 sensor is adhered to the patient’s neck.
“VO100 Sensor is disconnected”	<ul style="list-style-type: none"> - The VenArt Cardiac Output is detecting that the VO 100 sensor is not connected to the isolator or the isolator is not connected to the computer. 	<ul style="list-style-type: none"> - Ensure that the VO100 sensor is correctly connected to the isolator and computer.
“USB Pulse Oximeter Sensor is disconnected”	<ul style="list-style-type: none"> - The VenArt Cardiac Output is detecting that the Pulse Oximeter sensor is not connected to the computer 	<ul style="list-style-type: none"> - Ensure that the Pulse Oximeter sensor is correctly connected to the computer.
“Please Wait”	<ul style="list-style-type: none"> - The VenArt Cardiac Output is loading 	<ul style="list-style-type: none"> - Before continuing wait until this message disappears

For Technical Assistance

In the event that your problem is not resolved with the help of Table or if you have additional problems to report, contact your distributor or Mespere LifeSciences directly.

Section 7: Specifications

Performance

Table 3: Performance specifications

Parameter	Specification	Notes
VO 100 Jugular Venous Oximetry SjvO ₂ Measurement Range Accuracy of SjvO ₂ Resolution of SjvO ₂	0 – 100% ± 2.7% 1%	1,2
Nonin 8000S SpO ₂ Measurement Range Accuracy of SpO ₂ Resolution of SpO ₂ BPM Measurement Range Accuracy of BPM Resolution of BPM	0 – 100% ± 2% 1% 0 – 300BPM ± 3BPM 1BPM	
Contec CMS50E SpO ₂ Measurement Range Accuracy of SpO ₂ Resolution of SpO ₂ BPM Measurement Range Accuracy of BPM Resolution of BPM	0 – 100% ± 2% 1% 30 – 250BPM ± 2BPM 1BPM	
CO Measurement Range Resolution of CO	0.0 – 20.0 L/min 0.1 L/min	
SV Measurement Range Resolution of CO	0 – 300mL 1mL	
SVV Measurement Range Resolution of SV	0 – 200% 1%	

NOTES:

- 1** Measurement range is 45% - 91%
- 2** Accuracy is based on a validation study comparison with external jugular venous blood samples and a Co-Oximeter on healthy volunteers. In the SjvO₂ range of 45-91%, the accuracy was ± 2.7% in the study STP-9100001 at the University of California San Francisco, UCSF (San Francisco, California, USA).

Electrical

Table 4: Electrical specifications

Parameter	Specification	Notes
Computer Specifications	CPU: Quad Core Intel® Atom™x7-Z8700 processor (2MB Cache, 1.6GHz with Intel Burst™ technology up to 2.4GHz) or faster RAM: 2G or more	

	Storage: 64GB or more USB: 5Vdc, 500mA, USB 2.0 port OS: WIN 10		
Power Supply	AC Input: 100-240V, ~0.3A, 50-60Hz DC Output: 5.2V \approx 2.5A		
Measurement NIR wavelength	905 \pm 5 and 660 \pm 3nm		
Base Sampling Modulation Rate	2 kHz		
NIR LED duty cycle	7.5 ms		
NIR power	Peak	14 mw	3,4
	Average	0.9 mw	

NOTES:

- 3 Peak power indicates instantaneous electrical power dissipated by one NIR LED when 'on'. Average power reflects the time-averaged electrical power dissipated by one NIR LED when the system is running normally at a sampling rate of 2kHz and an NIR LED duty cycle of 9.1%.
- 4 Power figures are given for one NIR LED. Only one of the two NIR LEDs is active at any given time.

NOTE



The Computer and the device form a ME system.

Environmental

Table 5: Environmental specifications

Parameter		Specification
Temperature	Operating	10 °C – 38 °C
	Transportation/Storage	-20 °C – 50 °C
Humidity	Operating	30% – 70%
	Transportation/Storage	20% – 80%
Air Pressure	Operating	86 – 106 kPa
	Transportation/Storage	86 – 106 kPa

Physical Characteristics

Table 6: Physical characteristics specifications

Parameter	Specification
Data Display	SjvO ₂ , SpO ₂ , BVI, CO, SV, SVV plethysmographic waveforms, and corresponding trends
Display update rate	1 second

Compliance

Table 7: Compliance

Parameter	Specification
Safety Standard for Medical Equipment	IEC 60601-1 edition 3 Amendment 1
EMC Standard	IEC 60601-1-2 Class A
Degree of protection	Type BF-Applied part
Mode of Operation	Continuous

Warranty

Mespere LifeSciences Inc. warrants to the Purchaser for a period of one (1) year from the date of purchase for the monitor and software. Mespere warrants that the Product delivered is free from defects in workmanship or materials and the Product will perform as labelled in the directions for use. Mespere LifeSciences will work with the end-users distributor, or directly with the end user to determine warranty coverage for issues, should they arise, and if the issue is not covered by warranty, will provide a quote with respect to the repair costs to rectify the issue.

EXCLUSIONS

The warranty does not apply in the following conditions:

- 1) The product is modified without authorization from Mespere LifeSciences Inc.
- 2) Used with devices not supplied or authorized by Mespere LifeSciences Inc.
- 3) Used with sensor not supplied by or approved by Mespere LifeSciences Inc.
- 4) Used in conditions other than mentioned in the Instructions For Use
- 5) Products used for testing or demonstration purposes
- 6) Mishandling of equipment
- 7) Accident, fire, water, vandalism, weather or any act of God

Manufacturer's Declaration

WARNING



The use of accessories and cables other than those specified with the exception of accessories and cables qualified and sold by Mespere LifeSciences Inc. may result in increased emissions or decreased

immunity of the equipment and may cause the system to be non-compliant with the requirements of IEC 60601-1-2:2014.

Table 8: Electromagnetic Emissions

The VenArt Cardiac Output is intended for use in the electromagnetic environment specified below. The customer or the user of the VenArt Cardiac Output should assure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic Environment Guidance
RF emissions CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The equipment is suitable for use in all establishments other than domestic, and may be used in domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes, provided the following warning is heeded: Warning: This equipment/system is intended for use by healthcare professionals only. This equipment/system may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures, such as re-orienting or relocating the equipment or shielding the location.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

Table 9: Electromagnetic Immunity

The VenArt Cardiac Output is intended for use in the electromagnetic environment specified below. The customer or the user of the VenArt Cardiac Output should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±4 kV contact ±4 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines Input/output lines are exempted because none are more and 3 meter in length	Mains power quality should be that of a typical commercial/residential or hospital environment.
Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial/residential or hospital environment.

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 U _T = 240 Vac and 120Vac	<5 % U _T (>95 % dip in U _T) for 0.5 cycle 40 % U _T (60 % dip in U _T) for 5 cycles 70 % U _T (30 % dip in U _T) for 25 cycles <5 % U _T (>95 % dip in U _T) for 5 sec	<5 % U _T (>95 % dip in U _T) for 0.5 cycle 40 % U _T (60 % dip in U _T) for 5 cycles 70 % U _T (30 % dip in U _T) for 25 cycles <5 % U _T (>95 % dip in U _T) for 5 sec	Mains power quality should be that of a typical commercial/residential or hospital environment. If the user of the equipment requires continued operation during power mains interruptions, it is recommended that the equipment be powered from an uninterruptible power supply or a battery.
Power frequency (50 Hz/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial/residential or hospital environment.

Table 10: Electronic Immunity for Portable and Mobile RF Equipment

The VenArt Cardiac Output is intended for use in the electromagnetic environment specified below. The customer or the user of the VenArt Cardiac Output should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment Guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the equipment including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \frac{3.5}{v_1} \sqrt{P}$ $d = \frac{3.5}{v_1} \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \frac{3.5}{v_1} \sqrt{P} \quad 800 \text{ MH to } 2.5 \text{ GHz}$ where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

NOTE

At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey¹ should be less than the compliance level in each frequency range².

Interference may occur in the vicinity of known RF transmitting devices and equipment marked with the following symbol:



Table 11: Recommended Separation Distances

The ME equipment is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the ME equipment can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the ME equipment as recommended below, according to the maximum output power of the communications equipment.

Rated Maximum Output Power of Transmitter W	Separation Distance According to Frequency of Transmitter m		
	150 kHz to 80 MHz $d = \frac{3.5}{v_1} \sqrt{P}$	80 MHz to 800 MHz $d = \frac{3.5}{E_1} \sqrt{P}$	800 MHz to 2.5 GHz $d = \frac{7}{E_1} \sqrt{P}$
0.01	0.12	0.12	0.24
0.1	0.37	0.37	0.74
1	1.17	1.17	2.34
10	3.69	3.69	7.38
100	11.7	11.7	23.4

¹ Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the equipment is used exceeds the applicable RF compliance level above, the equipment should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the equipment.

² Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE



At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE



These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Table 12: Cable Compliance

The following cables comply with:

- RF emissions, CISPR 11, Class A/Group 1
- Electrostatic discharge (ESD), IEC 61000-4-2
- Radiated RF, IEC 61000-4-3
- Electric fast transient/burst, IEC 61000-4-4
- Conducted RF IEC 61000-4-6

Section 8: Appendices















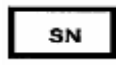


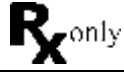
Appendix 1: List of Abbreviations




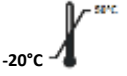

Table 13: Glossary of Abbreviations

Acronym	Meaning
SjvO ₂	Venous Blood Oxygenation from jugular vein This is the oxygen saturation of the venous blood in the jugular vein.
SpO ₂	Arterial Blood Oxygenation from finger This is the oxygen saturation of the arterial blood determined by the finger pulse oximeter.
EJV	External Jugular Vein This is the vein in the side of the neck in which the Neck Patch is placed.
IJV	Internal Jugular Vein As differentiated from the EJV used by the Mespere VO 100 System.
BVI	Blood Volume Index The sum of the amplitude attenuation of the two wavelengths. It is the blood to tissue volume ratio of regional tissue. For example; if BVI is 10% that means that blood fills 10% of the tissue volume.
CO	Cardiac Output The amount of blood the heart pumps through the circulatory system in a minute (L/min).
CI	Cardiac Index The amount of blood the heart pumps through the circulatory system in a minute per body surface area (L/min/m ²).
SV	Stroke Volume The amount of blood ejected from the left ventricle with each heartbeat in millilitres (mL).
SVV	Stroke Volume Variation The change in stroke volume with each heartbeat in percentage (%).
NIR	Near-Infra-Red Light having a wavelength between 0.7- and 5-microns (700nm to 5000nm).

Appendix 2: List of Symbols

Table 14: Glossary of Symbols

Symbol	Meaning
	“Note” message used to convey tips, helpful information and instructive statements.
	“Caution” message used to call attention to installations, practices, and operations that, if not properly performed or adhered to, could result in damage to the equipment.
	“Warning” message used to call attention to installations, practices, and operations that, if not properly performed or adhered to, could result in personal injury to the patient and/or user.
	Mark of conformity to European Union Medical Device Directive.
	Federal law (U.S.) restricts this device to sale by or on the order of a physician.
	Direct current (DC)
	Power indicator, located onto or near power switches.
	ESD sensitivity. This symbol may appear adjacent ports on the equipment that have heightened sensitivity to static discharge.
	Type BF applied part. This symbol indicates that the applied part is suitable for connection to the patient and that it delivers Type BF isolation.
	Consult Manual
	Class II Symbol
	European Authorized Representative
	Lot Number
	Reference/Model Number
	Serial Number
	Part number (re-order number)
	Manufacturer
	Consult Instructions For Use

Symbol	Meaning
	Single Use Only
	Product is to be taken to separate collection at end of product life. Do NOT dispose of the product as unsorted municipal waste.
	Date of Manufacture
	Storage temperature range: -20°C to 50°C
<p data-bbox="293 657 370 684">NOTE</p> 	Not all symbols shown in Table may appear on the equipment or in this manual.

Appendix 3: List of VenArt Cardiac Output Components

Table 15: List of VenArt Cardiac Output Components

Description	Component Part Number
Display Software	OS-VenArt
Instructions for Use – English	IFU-VenArt
Quickstart Guide – English	QG-VenArt

Contact:

Mespere LifeSciences Inc.
180 Frobisher Drive, Suite 1C
Waterloo, Ontario
N2V 2A2
Tel: (519) 884-7575
Fax: (519) 884-8810
Email: info@mespere.com
www.mespere.com

Authorized Representative:



Ms Lu Lu
Karl-Jatho-Weg 10,
71034 Boeblingen, Germany
Phone: +0049 17695706210

VenArt-IU_EN Rev F
Date of issue 2020-11
Printed in Canada

© 2020 Mespere LifeSciences Inc.

All rights reserved. No parts of this publication may be reproduced, in any form whatsoever, without the prior written permission of Mespere LifeSciences, Inc.

